

BARRIER MASK USAGE


PUTTING ON & TAKING OFF THE MASK



BEFORE PUTTING ON A MASK, **CLEAN HANDS WITH SOAP AND WATER** OR ALCOHOL BASED HAND DISINFECTANT.



INSPECT MASK BEFORE PUTTING IT ON. MAKE SURE IT IS **FREE OF DIRT AND GRIME**, AND NOT DAMAGED.

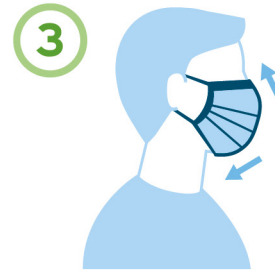

COVER MOUTH AND NOSE WITH MASK AND MAKE SURE THERE ARE **NO GAPS BETWEEN YOUR FACE AND THE MASK**.



IF THE MASK HAS A METAL NOSE PIECE, USE YOUR FINGERS TO **PRESS DOWN & FORM IT TO YOUR NOSE**.



AVOID TOUCHING THE MASK WHILE USING IT; IF YOU DO, CLEAN YOUR HANDS WITH SOAP AND WATER OR ALCOHOL BASED HAND DISINFECTANT.



REMOVAL OF MASK: REMOVE FROM BEHIND (DO NOT TOUCH FRONT OF MASK). CLEAN HANDS BEFORE AND AFTER WITH SOAP AND WATER OR ALCOHOL BASED HAND DISINFECTANT. IF DISCARDING THE MASK, PLACE IN A CLOSED BIN.

REUSAGE OF MASKS

DO NOT RE-USE SINGLE-USE MASKS.

INSPECT THE MASK PRIOR TO EVERY USE. IF THERE ARE ANY VISIBLE TEARS, THE MASK IS DIRTY, OR THE MASK IS HARD TO BREATHE THROUGH, IT SHOULD BE DISCARDED & REPLACED.



THE BARRIER MASK **SHOULD BE STORED BETWEEN USES** IN A CLEAN BAG.